PET DEVICE 600

REF 10280/PET

PET DEVICE 1200

REF 10282/PET

PET DEVICE 2400

REF 10283/PET

PET DEVICE 4800

REF 10284/PET

**DIESSE Diagnostica Senese
S.p.A.**

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PET DEVICE

(EN)

For the recharging of erythrocyte sedimentation rate tests performed on MINI-PET instrument.

1. INTENDED PURPOSE

Dedicated tube with Transponder RF.

To be used only once for the recharging of erythrocyte sedimentation rate tests performed on MINI-PET instruments.

2. INTRODUCTION AND PRINCIPLE OF THE METHOD

The insertion of PET Device permits the recharging of the instrument for the number of tests indicated in the **REF**.

3. PRECAUTIONS

Once used the PET Device must be disposed of in accordance with the regulations concerning electronic devices.

Warnings

The PET Device can only be used once. After being used for the recharging, the instrument will not recognize the tube again. Once performed the recharging, tests are valid for a year.

4. CONTENT

REF	10280/PET	1 tube containing 600 tests
REF	10282/PET	1 tube containing 1200 tests
REF	10283/PET	1 tube containing 2400 tests
REF	10284/PET	1 tube containing 4800 tests

5. PROCEDURE

The instruments are provided with a RF (radio frequency) reader that reads the data contained in the transponder and updates the internal meter recording the number of tests that can be performed. This number corresponds to the number of tests indicated in the PET Device code number.

For the recharging procedure refer to the Operating Manual of the instrument.

6. LIMITATIONS

The PET Device can be used on MINI-PET.

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